

INSTRUCTIONS FOR USE

CAUTION

To protect your health and that of others, it is very important to follow these instructions for use.

MASK UNS 1

Area of use : This mask is intended for the professional or general public and is exclusively reserved for Non Sanitary Uses (UNS). It is intended to prevent droplet projections. The use of this mask is part of the strict application of measures related to containment, measures of work organization as well as barrier gestures. This collective mask is intended to protect an entire group wearing these masks in the context of their professional activity. This mask is worn by people with occasional professional contact with other colleagues (business, service, office). **It is not intended for use by caregivers in contact with patients.** This mask can be indifferently offered to professionals or the general public. This mask does not replace barrier gestures (regular hand washing, physical distance, reduction of contact with other people).

Performances : Test protocol described in the DGA document of March 25, 2020 (test report RP / 20-3564 n ° 2000305 of April 10, 2020)

Category 1 mask exclusively reserved for non-sanitary uses according to the interministerial information note of March 29, 2020 updated on April 26, 2020

Particle filtration efficiency of 3 µm: > 90%

Breathability: air permeability test > 96 L.m2.S-1 for a vacuum of 100 Pa

Material whose performance has been measured by the laboratories of DGA Maîtrise NRBC, 5 rue Lavoisier, 91710 Vert-le-Petit (report RP / 20-3564 / DGA MNRBC / 2000305 / NP Version 1 of 15/04/2020) and which supports 10 washes).



Recommendations and limits of use: the wearing time of the mask is limited to 4 hours. Test to wear for 4 hours carried out by us. Wash before first use.

Mask handling recommendations:

Before putting on the mask:

- before putting on a mask, wash your hands with soap and water or a hydroalcoholic solution;
 - inspect the mask and make sure there are no holes, tears or damage.
 - It is recommended to wear the mask on bare skin, avoiding contact with hair.
 - Do not modify the mask in any way.
- To put on the mask:

- Hold the mask by elastic straps
- apply the mask at the level of the mouth so as to cover the nose and mouth and make sure it fits as well as possible on the face; put the elastic straps behind the ears; check that there is no air jet in the eyes during a strong exhalation;

When you wear a mask :

- avoid touching it; do not move the mask.
- do not wear it in the waiting position on the forehead or on the chin
- each time you touch a used mask, wash your hands with soap and water or a solution hydroalcoholic ;

To remove the mask:

- wash your hands with soap and water or a hydroalcoholic

solution, unhook the elastic straps to peel off the face mask; remove it from behind (do not touch the front of the mask); immediately throw it in a closed bin or isolate it in a plastic bag while waiting to wash it; wash your hands with soap and water or with a hydroalcoholic solution.

You have to change the mask:

- when the mask has been worn for 4 hours
- when you want to drink or eat
- when it becomes difficult to breathe
- if the mask gets wet or gets wet by the weather
- if the mask is damaged
- if the mask is deformed and no longer fits properly against your face.

Mask maintenance: Before the first use, the mask must be washed. After use, the mask must be isolated either to be thrown away or to be washed. It is recommended to isolate the mask in a disposable plastic bag or in a reusable washing bag. In the case of the reusable bag, mask and bag can be washed together and in this case, remove the mask from the bag.

Explanation on the washing method: Washing in a household machine with a detergent suitable for the fabric, the cycle of which will include at least a 30-minute tray at 60 ° C. Drum drying.

This mask has been designed to be reused and washed 10 times. After 10 washes, the filtration of the mask is no longer guaranteed. Mechanical drying or conventional drying followed in both cases by steam ironing.

Other information concerning the interview: do not put alcohol or bleach

Storage: unused masks to be stored in a clean, dry place in a clean, airtight container away from light.

Context of use: this mask is strictly adapted to the health crisis linked to COVID 19.

Material composition: External 100% Polyester. - Internal 100% Cotton

Warning : Despite all the care taken in the manufacture of these masks, it is the sole user and in no case the manufacturer or supplier to ensure the adequacy of the mask with the characteristics of the environment of use. The recommendations and methods of use described above should be scrupulously followed. Masks should not be worn by children under the age of 4. Retail sale authorized as long as each mask includes a notice. Please note, if you are sick, this mask is not suitable. Ask for your doctor's opinion. As part of the workstation risk analysis, the user must check that the actual breathability is compatible with the particularities of the workstation and in particular the effort in application of article R.4323-91 of the Code work.



MASK USE ACCOUNT

The masks can be washed 10 times, so each mask can be used 10 times. To help you monitor and manage the usage count for your masks, please complete the following table by checking the boxes:

	use 1	use 2	use 3	use 4	use 5	use 6	use 7	use 8	use 9	use 10
Mask 1										

MASQUE UNS 1

Reference	TOUR DE COU LISSE + JERSEY COTON	
Manufacturer's Name	DATOVA sas	
Manufacturer's address	4 rue de la république - 69001 LYON	
Performances		
Mask type	3 µm particle filtration efficiency	Breathability
<input checked="" type="checkbox"/> Mask UNS 1	>90%	>96 l.m- 2s-1
<input type="checkbox"/> Mask UNS 2	>70%	>96 l.m- 2s-1